

Installation of WinCPRS

WinCPRS should be installed from the installation CD. Either run the program *Setup.exe* directly on the CD or using the Windows Run command on the Start menu. During installation some settings are asked for, normally the defaults are most suitable.

When you start the program *first* time, you get following message:

Can not find the license key or the key driver has not been installed. See the help page: Miscellaneous/License key.

This is normal situation since the drivers for the license key have not been yet installed. You can install the necessary drivers by selecting the *Help* menu and choosing the item *Install License Key*. This starts another installation procedure.

To use *WinCPRS* the license key must be all time connected to the printer port of the computer or the USB-type of key to its own connector. If the printer is needed, it can be connected to the license key.

***WinCPRS* software can be used in Windows 95/98/ME/NT4.0/2000/XP systems. To install the software in Windows NT4.0/2000/XP you need either Administrator or Power User rights.**

USB-type of license key cannot be used in Windows 95 or NT4.0.

If this software is used in multi-user environment, or the software is started from another computer over the network, the database functions of the software assume that the user has rights to modify the database related configuration file PDOXUSR.NET. This file can be normally found on the root directory of the C-drive. The location of this file can be checked by running the program BDEADMIN.EXE which can be found from the installation directory of *WinCPRS*. On the *Configuration* tab sheet of this program (on the left field of the program) there is directory *Configuration\Drivers\Native\Paradox* which has the property NET DIR (on the right field). This property tells the directory path where the configuration file can be found.

Absolute Aliens Oy

**Korjasmäenkatu 16
20360 Turku, Finland**

www.absolutealiens.com
support@absolutealiens.com



0537